


ACTIVESHOP[®]

INSTRUCTIONS FOR USE

YOSHIDA 5-in-1 HYDRO CLEANING DEVICE
Model - H5020

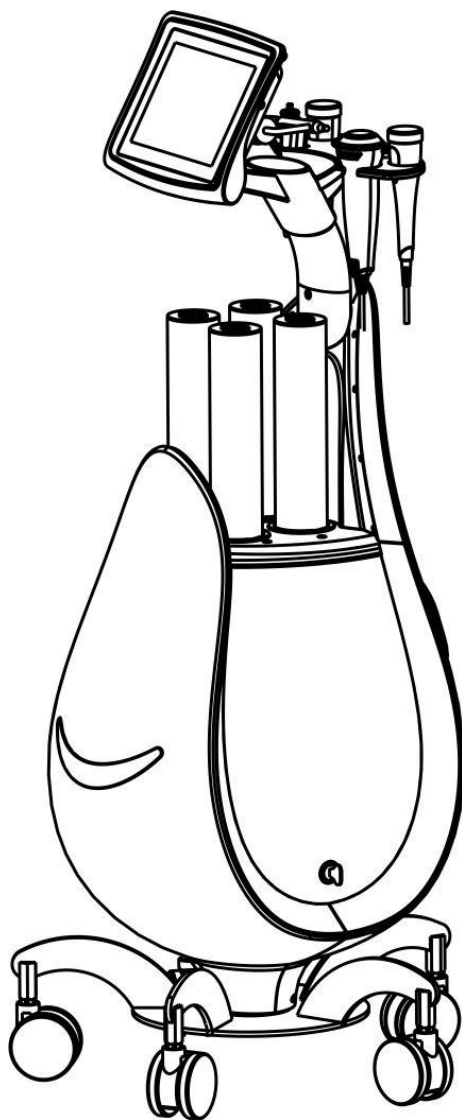

Importer:

ACTIV Tomasz Pacholczyk
ul. Graniczna 8b, building
DC 1 54-610 Wrocław,
Poland
www.activeshop.com.pl














This is not a medical product, please do not use it for medical treatment!

Specifications and technical data:

Product name	5 IN 1 Yoshida Hydrogen Peeling.
Item No.	H5020
Features	Oxybrasion AND hydrogen peeling、 Ultrasound、RF、spray、hammer heat - cold
Rated voltage/power	AC 100-120V ~
	AC 220-240V ~
Operating frequency	50/60Hz
Rated power	190W
Dimension (LXWXH)	61*41*90.1cm
N.W.	50Kg


Packing List:

Name	Appearance of the element	Quantity
Machine		1 set
Power cord		1 pc
RF head		1 pc
Head ultrasound		1 pc
Hammer heat - cold		1 pc
Airbrush spray		1 pc
Head for oxybrasion AND peeling hydrogen		1 pc
Grinding head		1 set

Hard plastic head		1 set
Silicone head		1 set
User Manual		1copy

Device manual:

1. Display screen
2. Accessories
3. Bottles (A,B,C,D)
4. Key
5. Front panel
6. Accessory holder
7. Connection sockets
8. On/Off button
9. Fuse
10. Power socket

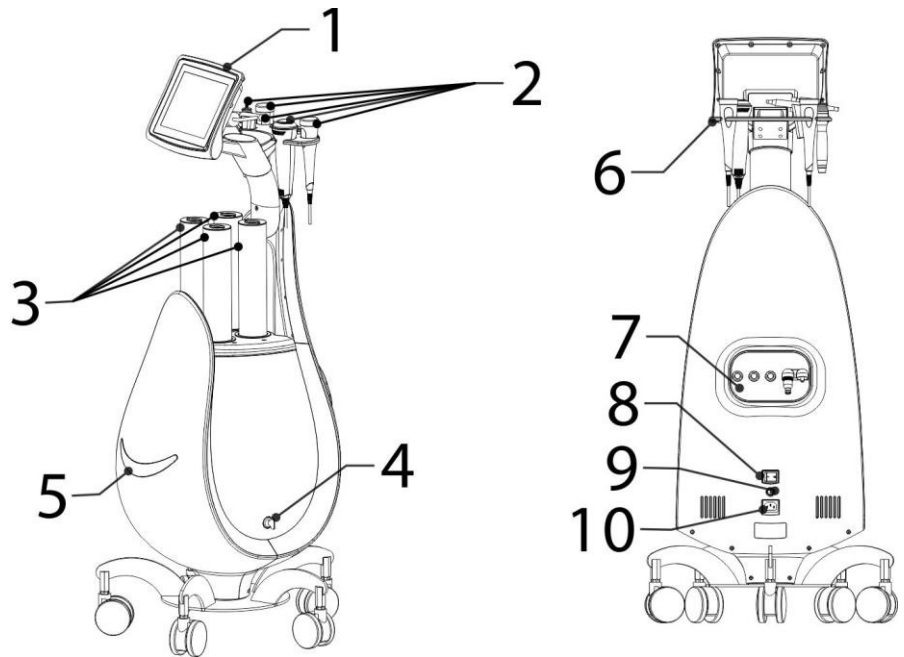

Instructions and comments :

1. Follow the instructions to install accessories.
- 2. Fill the container with mineral water or other liquid in water form, and then open the top button.**
3. Turn on the power switch on the side of the main frame. The power switch light will turn on.
4. Lift the display screen gently
5. Display and screen operation: Power on Power selection - various functions

6. Touch the display screen to select functions as needed Select

functions:

1. Aqua Clean (water and hydrogen purification)


2. Ultrasound

3. Spray

4. Settings

5. RF

6. Heat-cold hammer

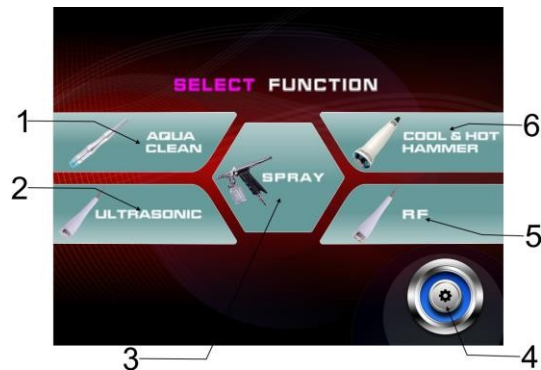

Features of the device

1. Aqua Clean (deep cleaning) .

2. Spray (application of active substances deep into the skin) .

3. RF (Lifting skin.) .

4. Ultrasound (application of active substances deep into the skin) .

5. Hammer warm-cold (stimulation of skin, pores AND blood vessels) .

Operational steps:

Aqua Clean function

By achieving negative pressure, ultra-small bubbles and nutrient fluids are fully combined. Specially designed small spiral suction heads work directly on the skin, keep the ultra-small bubbles in contact with the skin for a long time, increasing the peeling effect. They deeply cleanse the skin and remove aging keratin cells, removing sebum. Thoroughly


Remove all impurities, mites and sebum residues from hair follicles and replenish the skin with nutrients. The skin is moisturized, soft and shiny.

Preparation for use

Add special liquid or mineral water to the bottle



1. Press the button in the middle of the bottle.

2. Unscrew the top cover in the direction of the arrow.

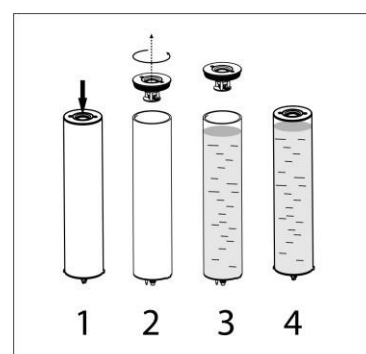

3. Add special liquid or mineral water to the bottle
4. Tighten the top cover in the opposite direction and press the middle button (see right image)
5. Insert the bottles into the appropriate host slots. (See image below)

Note: Before inserting, check that the bottom sealing ring is intact (see figure below). If the sealing ring is missing or damaged, it will leak! When inserting, check the small and large hole on the two sides to make sure match, you will not be able to insert it. Insert the 4 water bottles into the corresponding slots.

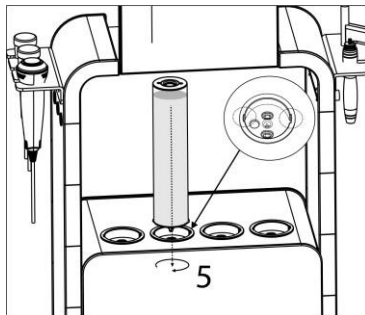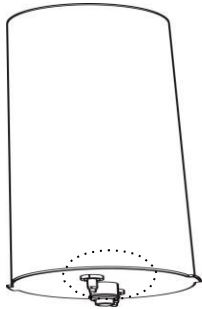


Directions for use

- (1) Select Aqua function screen.

MENU

1. Time reduction
2. Timer
3. Increase time
4. Return to main menu
5. Energy reduction
6. Energy bar

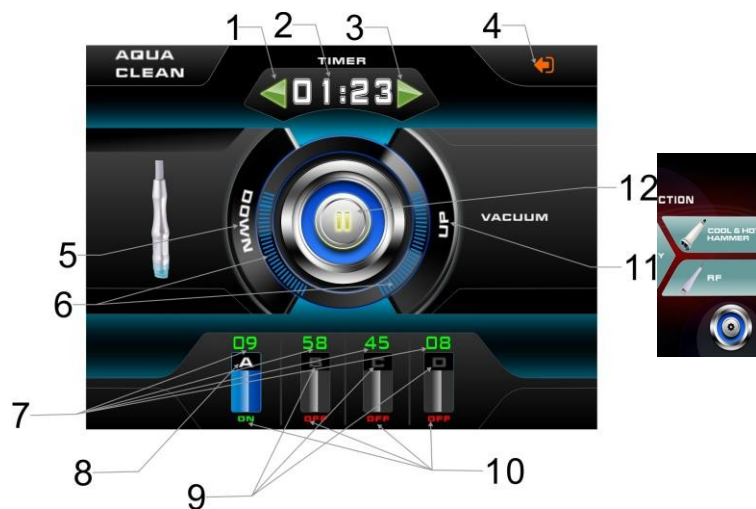

on

7. Percentage of water bottles used

8. Bottle A

(Enables the hydrogen-rich water function)

9. Bottle B,C,D

10. Working condition of the bottle


11. Increase energy

12. Start button

(2). Press button on center of lid bottles



(otherwise water may leak), then press Reduce Time




Increase the time  adjust the time of use, depending on the procedure to be performed (suggested 15-20min), press UP and DOWN to adjust the suction power as needed

(suggested increase from level 2), select the bottle you need, you can choose one or all, ABCD bottles are displayed on the screen, at the bottom of the active bottles are displayed

as ON  , inactive displays as OFF  , In bottle A can also be enabled

hydrogen water function, when it is in the function  click the bottle icon again  this means the hydrogenation function is on., when you want to disable this function,

just click on the cylinder icon again  this will close it. (Note : Only Cylinder A has this feature , B,C,D do not have this feature) .

Explanation: (Hydrogen water) according to scientific reports, hydrogen is insoluble in water using nanometer liquid-gas mixing technology, and uses physical methods to evenly surround hydrogen molecules around a water molecule to promote stable

A combination of hydrogen and water. It has the characteristics of high hydrogen concentration and good stability. Hydrogen is a colorless, odorless, non-toxic and odorless gas. The unique nature of hydrogen determines that hydrogen has many advantages in biology. The more obvious feature is the strong penetration of

layers of the skin, which can easily enter any part of the cell, such as the nucleus and

mitochondria, so hydrogen-rich water has a very strong antioxidant function.

Adjust the percentage of water per bottle, Simply click the number on the screen **4.5**, the number will appear at the top

1. Pin code selection window
2. Pin code selection keypad
3. Cancellation of pin code
4. Confirmation of pin code and exit




In the number entry interface, click the percentage value required to enter the above number, for example, enter 23, which means 23% of the water in the bottle is used.

press  the system saves the value and exits the interface. If the input is wrong,


Press  enter again.




After setting the required number, press  Start work.

(3) functions

Select the desired grinding head and mount it on the chuck

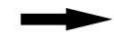

, Three grinding heads to choose from

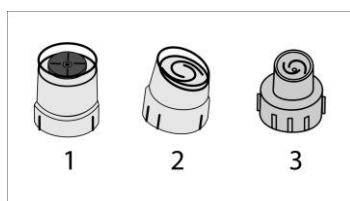

1. Grinding head
2. Hard plastic head
3. Silicone head


The grinding head is suitable for exfoliating dead skin. Hard plastic heads are suitable for regular use on the skin. Different models are used to clean the entire surface or parts of the face.

The silicone head is suitable for sensitive skin and does not irritate.

The entire face wash covers 15-20 minutes.

Notes on operations:

1. When bottle on used liquid is full, on screen appears will appear logo warning

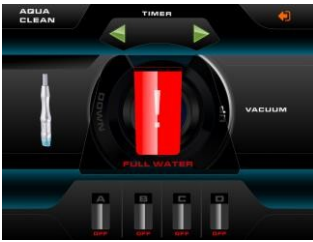

Then finish the work , open the rear door in the machine, remove the dirty water from the bottle. Following the direction of the arrow, clean the dirty water and install the bottle back, a

Then continue using it.

- 2, The water bottle sealing ring is damaged, it may cause leakage. If

accidentally leaks, immediately turn off the power,

water bottle, dry the water with a dry cloth and place it in a

A ventilated place until the device is dry.

reboot the system. Do not open the unit,

water is leaking! Otherwise, the device will be

destroyed!


- (3) When moving the device, press the

back to keep the water bottle closed to prevent spillage.

- 4, Keep the grinding head clean, clean the probe after each use and use alcohol or other disinfectant tools to sterilize or replace the new head.

- 5、 The treatment time should not exceed 15 minutes. People with a small area of skin on the face should have a shorter treatment performed. Too long a treatment may result in too much exfoliation or impaired absorption of active ingredients; the handpiece should not remain in one position for a long time to avoid damaging the skin.

- 6、 For too long an exfoliation treatment can damage the skin.

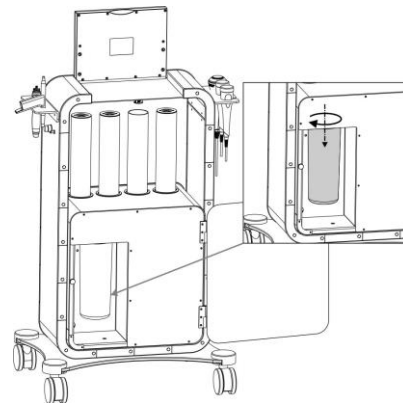

pull out

dry,

Try

when the


bottle

Oxygen water spray function

Performance: water or other substance to infuse, facilitates skin absorption, skin oxygenation, skin cell activation, skin nourishment.

Introduction to service

1. Drain (control the volume of oxygen ejection)
2. bottle
3. handle

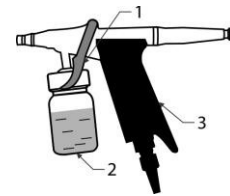


Prepare before use



Add pure water or essence to the

Instructions:

- (1) Selection The SPRAY icon when selecting a function goes to the spray interface.

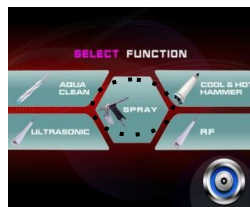

Interface introduction

1. adjustment time
2. launch the pause key
3. Exit the current function button



- (2) Increase or decrease to the correct time (recommended 15 minutes) When all settings are complete, press the button to start.

How to use

- After starting the corresponding function on the display, Hold the handle and gently pull the trigger. The mist will come out of the nozzle, pour different kinds of liquids into it; After changing to a different liquid, first stop the operation of the head, you need to CLEAN bottle to place another liquid in it. After spraying, pause the function infusion, Then using a cotton swab or soft paper to remove excess fluid from the client's face.

Notes on use:

- It is better to pull the trigger to the middle position to control the volume of mist, If you pull the trigger to the end, It is easy to cause too much water leakage. Return Pay more attention to important positions such as mouth, nose and eyes when performing the treatment, Ask clients to close their eyes and mouth, Spray mist from the client's forehead to the liquid from entering the clients' nostrils. Always keep a proper distance from the clients' face.


RF (radio frequency)

Functions: tissue heating, pore opening, collagen fiber length shortening,

Correct fine lines and wrinkles, restore skin elasticity and improve facial contours

Instructions:

- (1) Press RF in the function selection interface to enter

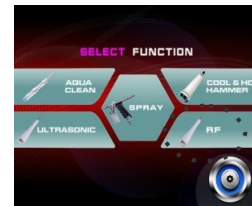



Interface introduction





1. Time adjustment
2. Energy bar (Shows the power output)
3. Energy reduction
4. Increase energy
5. Output waveform selection
6. Exit the current function button
7. Start / Pause button



energy)

- (2) increase  or decrease  to the correct time (recommended 30 minutes) ,

Then press decrease the energy or increase the energy to the correct height, Usually start to select the power on the second level, you should test it on your hand before starting the treatment on the client . The client may feel a slight warmth.

The output wave state has three modes to choose from,  continuous wave output state ,  state
mixed wave output ,  wide pulse output state 。 If necessary, derive the output wave state, generally start dialing  。

(2) Increase or decrease to the correct time (recommended 30 minutes), Then press decrease energy or increase energy to the correct amount, Usually start to select the power on the second level, you should test it on your hand before starting the treatment on the client . The client may feel a slight warmth.

Notes on use:

1. Apply a gel or radiofrequency preparation to the skin before the treatment, Do not use the RF head directly on dry skin, the RF head must be close to the skin, Do not let it leave the skins.
2. If the temperature is too high during use, just move it to other parts and move around faster, so the temperature will drop; If the temperature is not high enough, To move the RF head in the circle slower, The temperature will slowly rise.
3. It is not recommended for use during pregnancy or lactation. Not recommended for use on sensitive areas, skin with trauma, redness, inflammation, allergies or other skin conditions.
4. Remove metal accessories before starting treatment.

Ultrasound

Performance:

- 1, Stimulating blood vessels, eliminating redness: using pulse therapy
Ultrasound, cooperation with the appropriate product to activate blood vessels and cellular metabolism, and as a result of ultrasonic mechanical action, achieve heat effect and physical and chemical action, for Appropriate stimulation and massage, dilate blood vessels, stimulate blood flow and metabolic function, can Stimulate blood vessels to revitalize, eliminates the effect of erythema.
- 2, Anti-acne therapy with strong effect on acne: there are many forms of acne, it is
One of the more common cases with purulent condition blackheads, touch very painful, improper treatment tends to form hard scars. Ultrasound can affect the lymph nodes, blood and lymph circulation, accelerate the backlog of inflammatory cells.

- 3, Eliminate freckles and lighten spots: ultrasound can level freckles and hyperpigmentation, prevent their formation, weaken them and remove them with metabolism. Ultrasound can increase the absorption of skin care products (such as creams, masks, ampoules with brightening effect, etc.), Effect The removal of hyperpigmentation is visible after a series of treatments performed.
- 4, Decontamination and absorption: the mechanical massage function of ultrasound is used to regulate subcutaneous cells and increase cell membrane permeability on the one hand, on the other hand, enhances blood circulation and metabolism. In combination with proper blood circulation and Attaching care products, the effect is satisfactory.
- 5, Eliminate dark circles and puffiness of the eye, due to the effect of mechanical ultrasound massage, can accelerate blood and lymph circulation, and the effect of heating the tissue, can break up fat, accelerate the skin absorption of active ingredients, excessively accumulated moisture and fat are dissipated, puffiness and dark circles under the eyes disappear.

Instructions:



- (1) Press the ultrasonic button on the function selection interface to enter







Interface introduction

1. Time adjustment
2. Energy (shows energy output)
3. Reducing energy consumption
4. Increase in energy
5. Selection of output wave state
6. Exit the current function button
7. Start / Pause button



(2) To increase  or decrease  to the correct time (recommended 20 minutes) , Then press decrease energy or increase energy to the correct time

values, Usually choose the second level ,  Output wave state has three modes to choose , 
continuous wave output state ,  wide pulse output state. Choose the output wave state we need,
generally choose the corresponding parameter. .

Place the lotion or ampoule on the skin before using the ultrasound probe, Move the ultrasound probe in a circle from bottom to top.

Notes:

- 1, Keep the ultrasonic probe clean, clean it after each use and sterilize it with alcohol or

With another disinfectant.

2, Do not place the ultrasound probe on the eyeball.

3, Make you apply the right amount of active ingredient to the skin during the treatment.

4, In general, the treatment time should not exceed 15 minutes. Those with a small skin area should shorten the treatment time accordingly. The probe should not remain in a certain position for a long time to avoid damaging the skin tissue.

5, In the process of treatment, avoid prolonged use of ultrasound in the state of maximum energy expenditure, which will not only affect skin absorption, but may even damage skin tissue.

Hammer heat cold

Performance :

Cold therapy

1, Therapy good for narrowing blood vessels, sealing capillaries.

2, Strengthens elastin fibers, makes skin tight and elastic, smooths wrinkles, eliminates crow's feet.

3, blood vessels constrict, effectively reduce erythema and recirculate veins - it is simpler to cleanse and detoxify.

4, Reduce nerve sensitivity, have a calming effect, soothe the skin, swelling, pain, especially for allergic skin has a good supporting effect.

5, cold treatment+ blue light can relieve skin redness, anti-inflammatory effect


Heat therapy

1、 Slight heat, makes, relaxes muscles, balances nerve self-discipline, relieves pain. 2、 Stimulates blood circulation, increase the amount of oxygen in the blood, opens the pores, dilates the blood vessels and so that nutrients are absorbed faster.

3、 Increases the metabolic rate of the skin, increases the natural elimination of toxins.

Instructions:

(1) Select the hammer hot-cold function on the display press enter the display

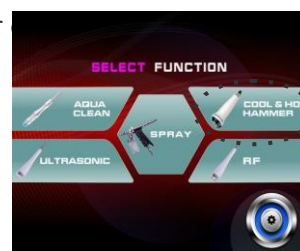

Interface introduction

1. Temperature setting
2. Start/Stop
3. Cooling the hammer
4. Hammer heating
5. Displays the current
hammer
6. Exit the current function button



temperature

(2) Performance

Effects Comments:

1. Cleanse your face with a makeup remover for best results.
 2. Do not let the hot and cold hammer wet. It should be wiped with a dry cloth.
 3. Please maintain proper heat emission, Not blocking heat emission holes
 4. This device is not intended for use by persons with poor physical fitness, slow reaction or mental disorders (and including children) unless used safely under direction or with the assistance of those responsible for the safety of the execution of the procedure.
 5. Children should be supervised so that they do not play with the equipment. Attention should be paid to the use of the entire machine:
1. When working, pay attention to the attachment and connection to the computer and do not pull excessively on the cables.
 2. If the power cord is damaged, replace it with a special cord or special components purchased from the manufacturer or its maintenance department.
 3. In the process of host operation, there are abnormal situations immediately turn off the power and Disconnect the power plug, contact your dealer to check, please do not open the case without permission.
 4. Do place or use in wet areas.
 5. Do not place flammable or explosive objects near it.

In case of discrepancy between the drawings in this manual and the actual items, the actual items shall take precedence. The company reserves the right to continuously update and product improvement. The above contents are subject to change without prior notice.

SECURITY MEASURES

1. The device is a professional equipment for use in beauty parlors. Operation of the device only by qualified cosmetic personnel allows to avoid undesirable consequences during the treatment.
2. Do not disassemble the device or attempt to use it for purposes other than those indicated in this manual

service. All service work and repairs must be performed only by an authorized service employee (ACTIV Tomasz Pacholczyk company).

3. Do not install or operate the device near liquids, wet places or with wet hands - so that no liquids get into the device.

4. Immediately remove the plug from the socket and contact the authorized service provided by ACTIV Tomasz Pacholczyk in the following cases:

a) liquid has entered the device, b) the device emits a strange odor, smoke or makes strange sounds, c) the power cord has been damaged, d) the device has been knocked over or its housing has been damaged.

5. Do not put any objects on the cables. Do not install the unit where someone may step on the power cord.

6. For safety purposes, unplug the power cord from the outlet each you finish working with the device.

7. Do not insert anything into the opening of the device. a foreign object gets the hole, remove the cord from the outlet and contact the distributor or manufacturer.

8. Do not place the device on an unstable cart, shelf or vehicle. If dropped, the device may be damaged.

9. In order to guarantee the established useful life of the device, it is recommended to keep an interval of one minute before pressing the power button again (between switching on and off and vice versa).

10. The power supply must comply with the specifications on the . Otherwise, there is a risk of incorrect operation of the device its damage or even burning.

NOTES:

1. In case of damage to the power cord or other malfunctions, contact an authorized service center - this will avoid danger.

2. This equipment may be used by children at least 8 years of age and by persons with impaired physical and mental capabilities and persons with inexperience and lack of familiarity with the equipment, if supervision or instruction is provided regarding the use of the equipment in a safe manner so that the associated risks are understood.

3. The device is not intended for play by children. Do not allow children to play with the device.

4. Do not leave the device unattended when children or persons without sufficient knowledge of how to use the device are in the vicinity.

5. This electrical appliance is not a toy. Improper use or contact with the electrical appliance may pose a health risk or lead to death, so use and store the appliance out of the reach of children.



DISPOSAL OF THE DEVICE

Dispose of used electrical appliances, fixtures and packaging to raw material reprocessing centers in accordance with environmental protection principles. Electrical appliances should not be thrown into household waste containers. Follow the rules for discarding this

type of equipment. In order to protect the environment in accordance with Directive 2012/19/EC on waste electrical and electronic equipment, waste electrical equipment should be collected separately, and sent to collection centers for waste electronic and electrical equipment.

At the end of their useful life, as well as old devices contain full-value recyclable components, so they should be handed over for reprocessing. Therefore, they should be handed over to appropriate collection points, as well as they can also be handed over to the ACTIV Tomasz Pacholczyk Company.

STORAGE AND TRANSPORT

The device should be transported at a temperature in the range of (-) 20 °C to (+) 50 °C only in its original packaging. Store the device in a well-ventilated room and a relative humidity of no more than 75%. Operating temperature from (+) 10 °C to (+) 40 °C. During storage and transportation processes, the device must not be thrown or dropped, exposed to mechanical shocks or atmospheric agents