

ACTIVESHOP[®]


INSTRUCTIONS FOR USE

Azzurro 803D electric cosmetic chair 3 motors



Importer:

ACTIVESHOP Ltd.
Ul. Graniczna 8b, building
DC2A 54-610 Wrocław,
Poland
www.activeshop.com.pl
[Rev. A/05/2024](#)

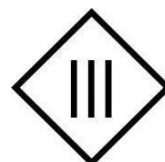


Before using the cosmetic chair, read the instructions and perform the following steps:


1. Make sure all packaging materials are removed.
2. Make sure the seat is in working order.
3. Place the cosmetic chair in the place where it will be used.
4. Plug the power plug into a suitable outlet.

Tip! Children and inexperienced people must not use this device. Children must be supervised and must not play near the cosmetic chair.

Dimensions of the device:

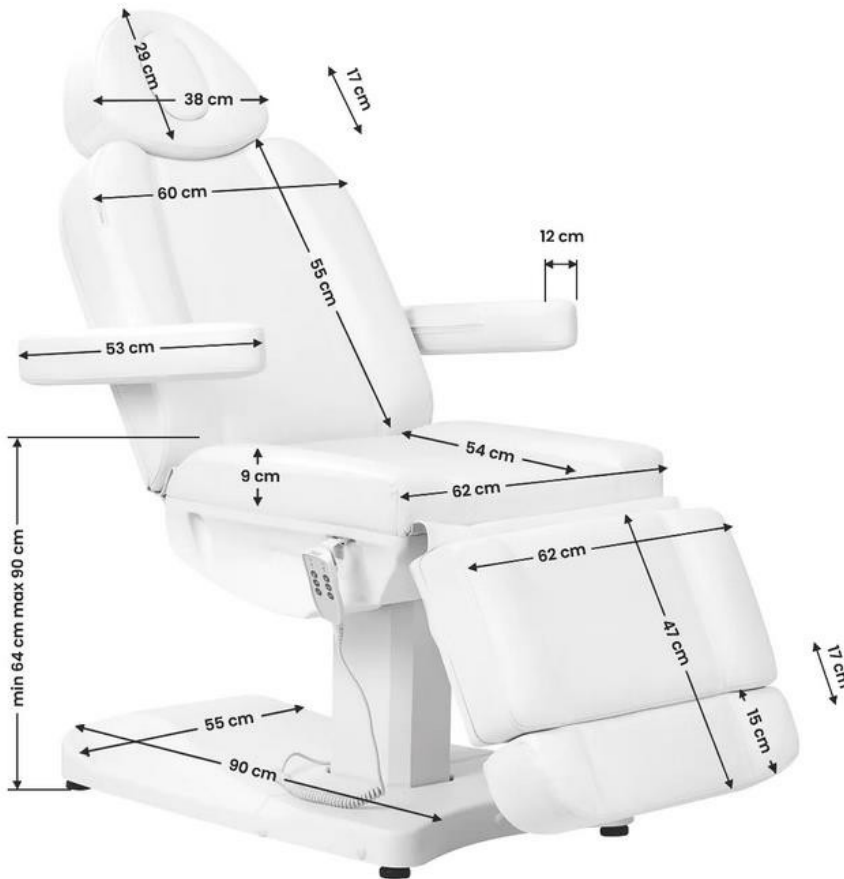

Construction of the device:

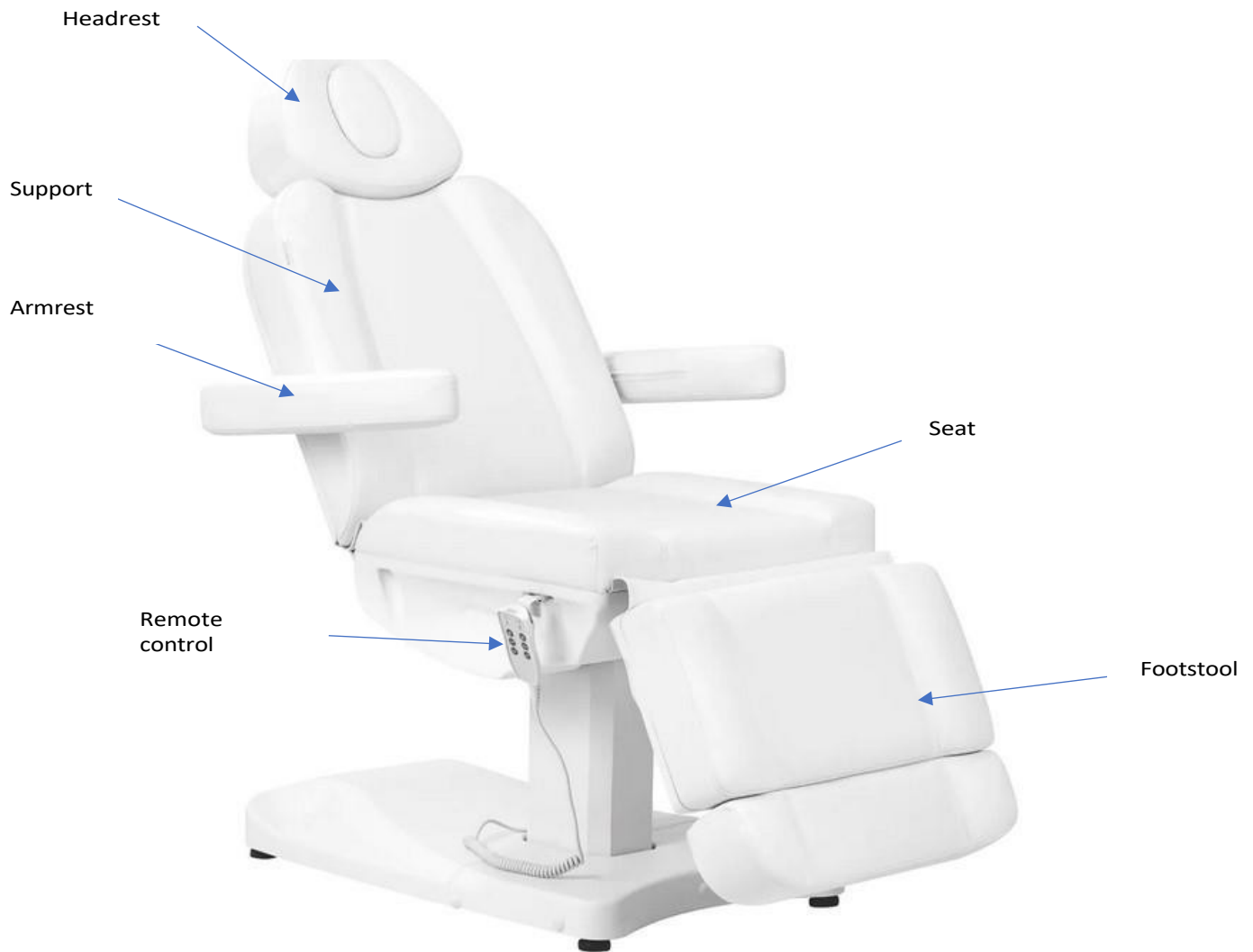

Warning:

1. The footrest is not designed to support the full weight of an adult.
2. Do not sit directly on the armrest and footrest.
3. Make sure that the distance between outlets does not exceed 2 meters.

Note


1. For safety reasons, always plug the power cord into a grounded outlet.
2. Do not tie the power cord to the unit. Lifting or pulling the power cord may damage it. Make sure that the power cord can be easily removed in an emergency.
3. Before moving the electric cosmetic chair, remove the power cord so that it does not get stuck between some parts of the chair.
4. If the power cord is damaged, immediately disconnect the cord, do not use the chair and contact an authorized service center. Use only the original power cord.
5. Do not let a person who does not perform the procedure control the control remote control.
6. Do not place any objects under the seat.
7. Make sure there is enough free space around the chair so that it can be folded and unfolded freely.

8. Make sure there are no sharp or harmful objects around the seat that could damage the device.
9. Do not modify the device yourself. If the chair does not work properly, stop using it and contact an authorized service center.


The chair has a manual control option with a remote control, as well as foot control options. According to your needs, you can adjust the height of the chair, the position of the backrest and the footrest.

Remote control functions:

1. Height adjustment of the seat (in the range of 64-90cm)
2. Backrest adjustment
3. Footrest adjustment

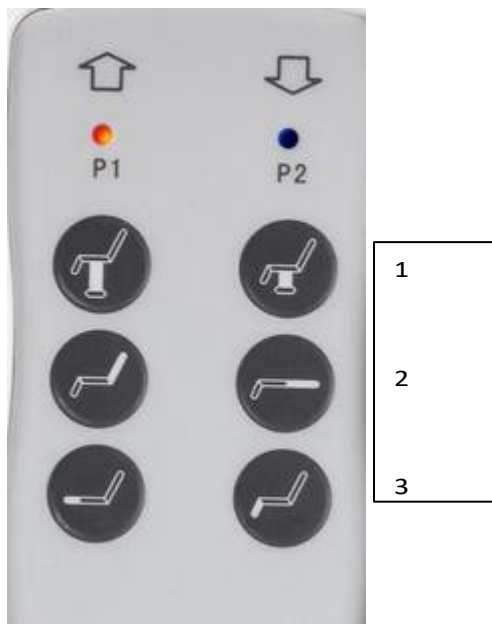


Foot control functions:

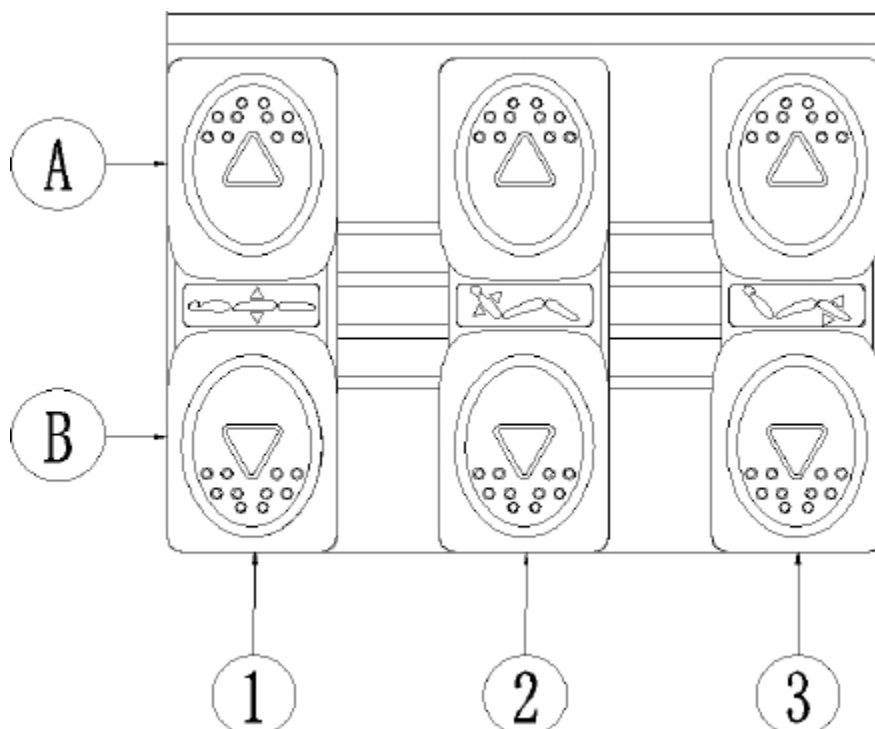

1. Adjusting the seat height. Press A to raise the seat or B to lower the seat.
2. Adjusting the backrest position. Press A to raise the backrest or B to lower the backrest.
3. Adjusting the footrest. Press A to raise the footrest or B to lower the footrest.

Footrest extension

The chair also has an option to extend the footrest, which makes it easier to perform treatments on taller people and improves comfort. The maximum length of the footrest extension is about 17 cm. To adjust the footrest, use the knob located on the bottom of the footrest

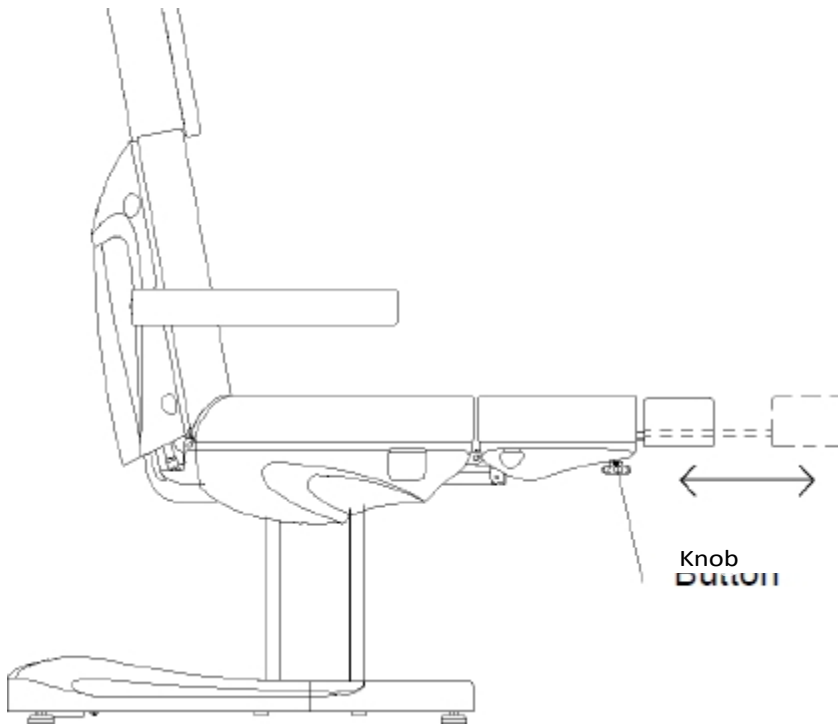


Armrest installation:

Slide the armrests into the hole located on the side of the backrest. The armrest should be firmly attached. Move it to the right or left so that it goes into place properly. Do the same with the other armrest. The armrest is perpendicular to the ground (regardless of the angle of the seat). It can be rotated 180°.

Headrest installation:

Slide the headrest into the hole located at the top of the seat back. When the headrest is at the correct height, tighten the crank located at the back to fix the headrest.

To lower or extend the headrest, unscrew the crank, adjust the headrest and tighten the crank.

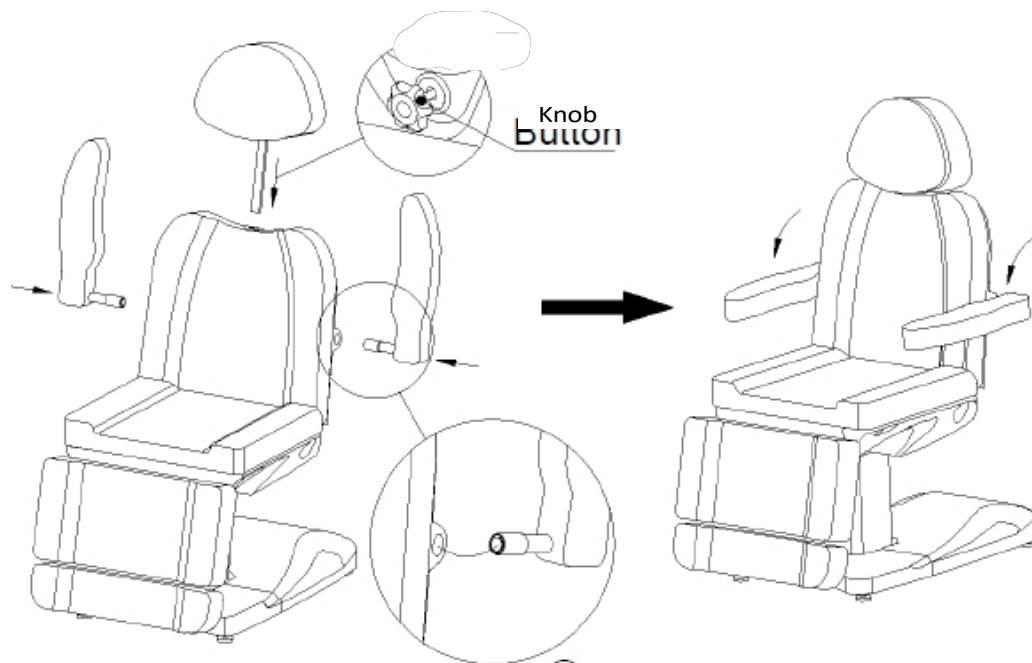

Cleaning and disinfection of the device:

Disconnect the device from the power supply before cleaning.

To keep the chair's surface in good condition, clean it regularly. Remember to clean and disinfect the chair after each use. It is recommended to perform a thorough disinfection once a month.

I Metal and plastic surfaces

To clean metal and plastic surfaces and controls wipe them with a damp cloth. To clean hard-to-reach areas or corners, a small brush will work well. Do not use too much liquid to clean the device. Then disinfect the device with a suitable liquid.

II Stainless steel surfaces

Cleaning with a mild detergent solution will work best. Ammonia and most solvents can be used to remove tough stains. Avoid chlorine-based solutions.

III Painted or chrome-plated metal surfaces

- Do not use water jets (showers, high-pressure water cannons) for cleaning.
- Do not clean in high temperature and high humidity, do not expose the device to conditions that may lead to liquid contamination.
- Do not use solvents and acids for cleaning.

IV Seat surface

- Wipe the surface of the seat evenly with a special leather cleaner, then wipe it with a clean, soft cloth.

Maintenance

Unplug the device before maintenance.

1. Repairs should be performed by experienced and manufacturer-authorized personnel. Repair by unauthorized personnel may cause injury or damage to the device, for which the manufacturer is not responsible.
2. Use only original spare parts approved by the manufacturer.
3. Do not use the electric beauty chair or accessories if they are not working properly.

- All maintenance and repair activities must be documented.

Livelihoods

Check the controls and their wiring once a week. Check the condition and function, of the following items at least once every six months:

- Power cord and its attachment
- Engine wiring
- Control box and its wiring
- Are the accessories attached
- Normal operation of the locking system

If you find any defects, stop using the equipment and contact an authorized service center. Only experienced or authorized personnel can open or replace the actuator / control unit.

If any part of the device is damaged, unplug the power cord and stop using it. Then notify the service center.

Troubleshooting

Problem	Cause	Solution method
No response after pressing buttons on remote control	<ol style="list-style-type: none"> The power button activates or not. Check that the pedal is depressed to the maximum. Check that the device is plugged in. Check that everything is connected correctly. 	<ol style="list-style-type: none"> Try restarting the power button. Try enabling other buttons and see if they work. Unplug the device and plug it in again. Disconnect the plugs and connect again.
The device makes strange sounds	Loose bolt	Tighten the screws

SECURITY MEASURES

1. The device is a professional equipment for use in beauty parlors. Operation of the device only by qualified cosmetic personnel allows to avoid undesirable consequences during the treatment.

2. Do not disassemble the device or attempt to use it for purposes other than those indicated in this manual. Any service and repair work must be performed only by an authorized service technician (ACTIVESHOP L t d.).

3. Do not install or operate the device near liquids, wet places or with wet hands - so that no liquids get into the device.

4. Immediately remove the plug from the outlet and contact the authorized service provided by ACTIVESHOP Ltd. in the following cases:

- a) liquid has entered the device, b) the device emits a strange odor, smoke or makes strange sounds, (c) the power cord was damaged, (d) the device was knocked over or its housing was damaged.


5. Do not put any objects on the cables. Do not install the unit where someone may step on the power cord.

6. For safety purposes, unplug the power cord from the outlet each time you finish working with the device.
7. Do not insert anything into the opening of the device. If a foreign object gets into the hole, remove the cord from the outlet and contact the distributor or manufacturer.
8. Do not place the device on an unstable cart, shelf or vehicle. If dropped, the device may be damaged.
9. In order to guarantee the established useful life of the device, it is recommended to keep an interval of one minute before pressing the power button again (between switching on and off and vice versa).
10. The power supply must comply with the specifications on the nameplate. Otherwise, there is a risk of incorrect operation of the device its damage or even burning.

NOTES:

1. In case of damage to the power cord or other malfunctions, contact an authorized service center - this will avoid danger.
2. This equipment may be used by children at least 8 years of age and by persons with impaired physical and mental capabilities and persons with inexperience and lack of familiarity with the equipment, if supervision or instruction is provided regarding the use of the equipment in a safe manner so that the associated risks are understood.
3. The device is not intended for play by children. Do not allow children to play with the device.
4. Do not leave the device unattended when children or persons without sufficient knowledge of how to use the device are in the vicinity.
5. This electrical appliance is not a toy. Improper use or contact with the electrical appliance may pose a health risk or lead to death, so use and store the appliance out of the reach of children.

DISPOSAL OF THE DEVICE

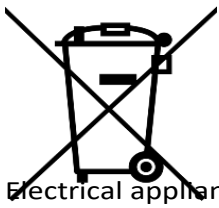

Dispose of used electrical appliances, fixtures and packaging to raw material reprocessing centers in accordance with environmental protection principles. Electrical appliances should not be thrown into household waste containers. Follow the rules for discarding this type of equipment. In order to protect the environment in accordance with Directive 2012/19/EC on waste electrical and electronic equipment, used

Electrical appliances should be collected separately and then transferred to points of

collection of waste electronic and electrical equipment. At the end of their useful life, as well as old appliances contain full-value r e c y c l a b l e components, so they should be handed over for reprocessing. Therefore, they should be handed over to appropriate collection points, as well as they can also be handed over to ACTIVESHOP Ltd.

STORAGE AND TRANSPORT

The device should be transported at a temperature in the range of (-) 20 °C to (+) 50 °C only in its original packaging. Store the device in a well-ventilated and relative humidity of no more than 75%. Operating temperature from (+) 10 °C to (+) 40 °C. During storage and transportation processes, the device must not be thrown or dropped, exposed to mechanical shocks or atmospheric agents.